ITALIANO

ENGLISH

ESPAÑOL

FRANÇAIS

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PERIPHERAL BLOOD SELECTIVE FILTRATION SYSTEM

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The **MONOCYTES MCT-A50SP00000** system is an assembly of class IIa and class IIb medical devices intended for the processing of human autologous peripheral blood to obtain mononuclear cells.

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IFU (Instructions for use)

The **MONOCYTES MCT-A50SP00000** assembled system contains the following medical devices:

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- 1 set of infusion procedures (SE4273SE)
- 1 filter for obtaining mononuclear cells (FL40/D27)
- 2 300ml bags for blood components sampling (4010005000)
- 1 50 ml bag for mononuclear cells collection (4010000000)

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1 - WITHDRAWAL Open the kit on the sterile field and mount it as shown on the label relating to the FL40 / D27 device.

Load the blood collection syringes with 10% anticoagulant (ACD-A); so 6 ml on 60 ml of blood. The recommended withdrawal is 60-120 ml. Clean the sampling area and aspirate the necessary volume of peripheral blood with a dedicated cannula needle.

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2 - PROCESSING Hang the kit from a stand, holding the upper bag up. Insert the blood taken through the upper connector to fill the same bag and activate the filter circuit by opening the dedicated clips. Wait until all the blood has passed through and completely escaped from the filter.

At this point the waste is deposited in the lower bag, while the cells remain trapped inside the filter.

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3 - RECOVERY Close the clips to exclude the filter circuit. Open the side clip. Aspirate sterile saline using the 10 ml syringe and connect the syringe to the lower side port. Gently inject sterile saline. Once the filter backwash operation is finished, recover the contents from the collection bag with a new syringe.

Dispose of the used filter, accessories, and needles according to the procedures indicated for the disposal of biologically contaminated material.

Use the adhesive labels for the traceability of the device in the appropriate forms.

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