

MONOCYTES
PERIPHERAL BLOOD MONONUCLEAR CELLS

Product Picture



Trading Name

MONOCYTES

Product Code

MCT-A30SP00000

Composed of:

MTK02650/HTA-03	INFUSION PROCEDURES SET
FL40/D27	FILTER FOR OBTAINING MONONUCLEATE CELLS
4010005000	BLOOD COMPONENT SAMPLING BAG 300 ML
4010000000	BAG FOR COLLECTING MONONUCLEATE CELLS 50 ML

Usò Previsto

MONOCYTES is a peripheral blood selective filtration system from which it is possible to obtain mononuclear cells. MONOCYTES works by gravity filtration, exploiting the force of gravity, avoiding any act that alters or stresses the cells themselves (centrifugation, cutting, etc.). The heart of the system is a membrane that selectively holds cells by size, and is capable of completely eliminating the inflammatory cell component.

CODICE	DESCRIZIONE	CE	CND	RDM
MCT-A30SP00000	KIT MONOCYTES	//	//	132880

Composition Kit Monocytes

CODICE	DESCRIZIONE	CE	CND	RDM
MTK02650/HTA-03	INFUSION PROCEDURES SET	0373	T0202	2138622
FL40/D27	FILTER FOR OBTAINING MONONUCLEATE CELLS	0425	B06	1556515/R
4010005000	BLOOD COMPONENT SAMPLING BAG 300 ML	0476	B019004	2142764/R
4010000000	BAG FOR COLLECTING MONONUCLEATE CELLS 50ML	0476	B019004	2142764/R

MONOCYTES (MCT-A30SP00000) IS AN ASSEMBLED KIT, THE CE AND THE CND OF REFERENCE ARE THOSE OF THE INDIVIDUAL CODES THAT COMPOSE IT (MTK02650/HTA-03, FL40 / D27, 4010005000, 4010000000).

Description of the individual devices

MTK02650/HTA-03	Description	Disposable sterile complete kit for infusion procedures
INFUSION PROCEDURES SET	Content	N ° 1 cloth 100x100 cm N ° 1 tray 14x24x2.5 cm N ° 2 hypodermic needles of 22 g 30 mm N ° 1 18g 2-way cannula needle L. 45 mm N ° 2 10 ml luer lock syringes N ° 2 syringes of 60 ml luer lock N ° 10 syringes of 3 ml luer lock
	Materials	All devices are CE marked. The technical specifications of the medical devices included in the kit are available on request.
	Packaging	With peel open bag in paper / PE suitable for sterilization.
	Sterilization	Ethylene oxide - validity 60 months
	CE Class	Ila sterile
	Produced by	BETATEX S.P.A. – Loc. Venella, Terentola di Cortona (AR)
FL40/D27	Description	System for the preparation of concentrated mononuclear cells
FILTER FOR OBTAINING MONONUCLEATE CELLS	Content	Male L / L connector and protective cap L / L female connector with bi-directional safety valve and protective cap NO DOP PVC pipe Sliding clamp in PE Y-piece in PVC Filter for the separation of mononuclear cells.
	Materials	All devices are CE marked. The technical specifications of the medical devices included in the kit are available upon request.
	Packaging	Medical paper bag suitable for sterilization
	Sterilization	Ethylene oxide - validity 60 months
	CE Class	Ila
	Produced by	MACROPHARM Srl – Caltanissetta (CL)

4010005000	Description	System for the preparation of concentrated mononuclear cells
BLOOD COMPONENT SAMPLING BAG 300 ML	Content	300 ML blood bag
	Materials	2 blood bags of 300 ML
	Packaging	All devices are CE marked. The technical specifications of the medical devices included in the kit are available on request.
	Sterilization	With peel open bag in paper / PE suitable for sterilization.
	CE Class	Ethylene oxide - validity 60 months
Produced by	Iib	

4010000000	Description	50 ML blood bag
BAG FOR COLLECTING MONONUCLEATE CELLS 50 ML	Content	1 50 ML blood bags
	Materials	All devices are CE marked. The technical specifications of the medical devices included in the kit are available on request.
	Packaging	With peel open bag in paper / PE suitable for sterilization.
	Sterilization	Ethylene oxide - validity 60 months
	CE Class	Iib
Produced by	MEDITALIA INDUSTRIALE Srl – Lovero V. (SO)	

Packaging
Monocytes System

- Each box contains:
- 1 SET OF INFUSION PROCEDURES (MTK02650/HTA-03)
 - 1 FILTER TO OBTAIN MONONUCLEATE CELLS (FL40 / D27)
 - 2 BLOOD COMPONENT SAMPLING BAG 300 ML (4010005000)
 - 1 BAG FOR COLLECTING MONONUCLEATE CELLS 50 ML (4010000000).

Each device inside the box is packaged in a double sterile, single-use bag.

Packaging labeling compliant with current regulations.

4 adhesive labels for traceability

The MONOCYTES Kit does not undergo any alterations whatsoever to normal storage conditions.

Warnings and Precautions

MONOCYTES contains single use sterile packaged products. Do not resterilize. Discard open, unused or damaged packages. Do not use in case of loss of sterility of the devices.

Keep the kit away from excessive heat sources. Keep at room temperature, away from direct heat sources, including sunlight.

The product must be stored in a cool and dry place.

Quality Monocytes Kit

All the components of the assembly are manufactured according to quality systems compliant with UNI EN ISO 9001: 2015 and UNI CEI EN ISO 13485: 2016 and Directive 93/42/CEE.

Assembler data

CLOVER ORTHOPEDICS SRL - Via Gadames 57/7 - 20151 Milano (MI)

Clover Orthopedics s.r.l.

Via Gadames n. 57/7, c.a.p. 20151 Milano

M. info@cloverorthopedics.com

W. cloverorthopedics.com

T. +39 02 457 902 31

F. +39 02 457 902 66

